

## Vitremer™ Core Buildup/Restorative System



### Dispensing


- Dispense an equal number of powder scoops and liquid drops.



1

#### Dispensing the powder

- Fluff the powder.
- Dispense level scoops.
- Dispense equal numbers of powder scoops and liquid drops

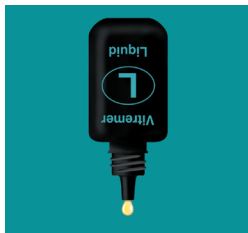

2

#### Dispensing the liquid

- Keep the tip of the vial clean.
- Hold the vial in a vertical orientation and squeeze.
- Avoid contacting the mixing pad with the tip.



3

#### Mixing

- Use a large cement spatula.
- First mix one-half of the powder into the liquid, then add the remaining powder.
- Mix aggressively.
- Continue mixing until all of the powder is incorporated into the liquid.
- Time the mixing. Do not mix longer than 45 seconds.
- Vitremer is designed to have a thick consistency.

**Vitremer™ Core Buildup/Restorative System**

---

**3M ESPE Customer Care/MSDS Information:**

**U.S.A. 1-800-634-2249 and Canada 1-888-363-3685**

3M, ESPE and Vitremer are trademarks of 3M or 3M Deutschland GmbH.  
Used under license in Canada. © 3M 2012. All rights reserved.

**3M ESPE**

Dental Products, St. Paul, MN 55144-1000 USA • [www.3MESPE.com](http://www.3MESPE.com)



44-0007-4945-5-C